SIZE=23X19 INCH, PRINT COLOR=BLACK

ClearSafe® Safety I.V. Catheter English

NOTE: U.S. Federal Law restricts this device sale by or on order of registered physician.

INTENDED USE: The MedSource ClearSafe® Safety IV Catheter is indicated to sample blood or administer fluids intravenously. The MedSource ClearSafe® Safety IV Catheter may be used for any patient population with consideration given to adequacy for the vascular anatomy appropriateness for the solution being administered and duration of the therapy.

The following instructions contain important information regarding the safe use of this product. Please read the Instruction For Use contents entirely before using this product. Failure to properly follow warnings and instructions could result in serious injury or death to the patient and/or clinician.

Product Description:

Each ClearSafe® Safety I.V. Catheter is comprised of the following components: protective needle cover, color-coded catheter hub, radiopaque catheter tube, teflon catheter holder, needle guide, medical grade stainless steel needle, needle hub, porous plug and gauge chamber.

The ClearSafe® Safety I.V. Catheter components combine for an ergonomic design that incorporates a safety feature. The safety feature of this device is intended to minimize accidental needle sticks. Please see the Instructions For Use section for further explanation of the safety feature.

The ClearSafe® Safety I.V. Catheter gauges are color coded for positive identification and range from 14G to 24G

ORANGE = 14G GREY = 16G GREEN = 18G PINK = 20G BLUE = 22G YELLOW = 24G

Warnings:

Clinicians must follow Instructions for Use. In addition to following the Instructions for use, clinicians must be trained to ensure proper usage of the device. Failure to do either of these could result in death or serious injury to the patient and/or clinician.

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Do not apply pressure during usage that causes the needle/catheter to flex.

In the unlikely event that the device does not function as described in the Instructions for Use, immediately dispose of the device in an approved sharps container.

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Cautions:

This device in an unopened package is sterile, non-toxic and non-pyrogenic. Discard if open, wet or damaged.

The materials used to manufacture this device do not contain natural rubber latex.

This product does not contain PVC.

Radiopaque I.V. catheter.

This device is for single use only. Any attempt to re-sterilize and/or recondition the device will compromise the integrity of the device and will have an adverse effect on performance.

US Federal Law restricts the sale of this device to or on the order of a registered physician.

Precautions:

This device is designed to reduce needle stick occurrences. Care must always be exercised when using this product. In addition to the Instructions for Use provided, it is strongly recommended that the clinician follow the guidelines put forth by the CDC and OSHA standards (USA) for blood borne pathogens when using this product.

During catheter insertion, maintenance and removal, follow current "Guidelines for the Prevention of Intravascular Catheter-Related Infections", Centers for Disease Control and Prevention, Atlanta, GA (USA), Institutional policy and procedures and professional standard of care.

Securely affix the catheter to the patient. Improper securing can result in loss of vascular access.

A firm connection between the catheter hub and administration set is necessary to prevent leaking.

The introducer needle is longer than the catheter tube and will enter the vein or artery first. This can cause a blood flashback prior to the entrance of the catheter tube into the vein or artery. Advancing the catheter and needle together slightly will ensure intravascular access for the catheter. When advancing slightly, it is recommended to reduce the angle of approach in order to minimize the chance of puncturing the posterior wall of the vessel.

In the event of an unsuccessful venipuncture, stabilize the catheter hub and actuate the device until it locks in the safety position. The safety position can be found by hearing a slight "click". Remove the catheter and properly dispose of both the spent catheter and locked Safety I.V. Catheter assembly.

Instruction for use:

Due to the Risk of Blood borne pathogen exposure, follow Universal Precautions during placement, use and removal of ClearSafe® Safety I.V. Catheter.

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Aspic technique, proper skin preparation and continued protection of the site are essential. Observe universal precautions on ALL patients.

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C. Insert needle into skin and vein and observe for blood flashback. The catheter is shorter than the needle. blood flashback may occur before catheter tip is fully in the vein. It may be necessary to advance the catheter and needle slightly to achieve full entry into the vein. To avoid puncturing the posterior vein wall, lower the needle until it is parallel to the skin.

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E. Keeping the device stable, retract needle housing until it "clicks" into place. This is the indication that the needle is now locked inside the needle guard.

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STERILE EO

Sterilised by Ethylene Oxide Gas

Caution, Consult
Accompanying Documents

Consult Instructions
For Use

Non-Pyrogenic

Do Not Use if
Package is Damage

Do not resterilize

Keep away
from sunlight

Keep dry

Temperature limitation (68°F-78°F
following OSHA guidelines for warehouses)

MEDSOURCE
www.medsourcelabs.com

Manufactured for:

MedSource
8600 Shelby Court
Chanhassen, MN 55317, USA
Made in India

AW/IFU_AU, Rev. 01
Issue Date: 06.09.21
N. Code: HR/DEVICE/MFG/MD/2019/000135

ClearSafe Comfort® Safety I.V. Catheter English

NOTE: U.S. Federal Law restricts this device saleby or on order of registered physician.

INTENDED USE: The MedSource ClearSafe Comfort® Safety IV Catheter is indicated to sample blood or administer fluids intravenously. The MedSource ClearSafe® Safety IV Catheter may be used for any patient population with consideration given to adequacy for thevascular anatomy appropriateness for the solution being administered and duration of the therapy.

! *The following instructions contain important information regarding the safe use of this product. Please read the instruction For Use contents entirely before using this product. Failure to properly follow warnings and instructions could result in serious injury or death to the patient and /or clinician.*

Product Description:

Each ClearSafe Comfort® Safety IV Catheter is comprised of the following components: protective needle cover, color-coded catheter hub, radiopaque catheter tube, teflon catheter holder, needle guide, medical grade stainless steel needle, needle hub, porous plug and gauge chamber.

The ClearSafe Comfort® Safety IV Catheter components combine for an ergonomic design that incorporate a safety feature. The safety feature of this device is intended to minimize accidental needle sticks. Please see the Instructions For Use section for further explanation of the safety feature.

The ClearSafe Comfort® Safety IV Catheter gauges are color coded for positive identification and range from 14G to 24G

ORANGE = 14G GREY = 16G GREEN = 18G PINK = 20G BLUE = 22G YELLOW = 24G

Warnings:

Clinicians must follow Instructions for Use. In addition to following the Instructions for use, clinicians must be trained to ensure proper usage of the device. Failure to do either of these could result in death or serious injuryto the patient and/or clinician.

Best practice for avoiding an accidental needle stick is to use the device as described in the Instructions for Use and discard immediately into an approved sharps container.

Do not apply pressure during usage that causes the needle/catheter to flex. In the unlikely event that the device does not function as described in the Instructions for Use, immediately dispose of the device in an approved sharps container.

Upon removal of the protective needle cover and prior to insertion, caution needs to be taken to ensure that the catheter does not come in contact with any foreign objects. Contact with a foreign object creates the potential for embolus.

Never reinsert the introducer needle inside the catheter. Reinsertion threatens the integrity of the catheter and could lead to embolus.

Cautions:

This device in an unopened package is sterile, non-toxic and non-pyrogenic. Discard if open, wet ordamaged.

The materials used to manufacture this device do not contain natural rubberlatex. This product is not made with PVC. This product is not made with DEHP. Radiopaque I.V. catheter.

This device is for single use only. Any attempt to re-sterilize and/or recondition the device will compromise the integrity of the device and will have an adverse affect on performance. US Federal Law restricts the sale of this device to or on the order of a registered physician.

Precautions:

This device is designed to reduce needle stick occurrences. Care must always be exercised when usingthis product. In addition to the Instructions for Use provided, it is strongly recommended that the clinician follow the guidelines put forth by the CDC and OSHA standards (USA) for blood borne pathogens when using this product.

During catheter insertion, maintenance and removal, follow current "Guidelines for the Prevention of Intra-vascular Catheter-Related Infections", Centers for Disease Control and Prevention, Atlanta, GA (USA), Institutional policy and procedures and professional standard of care.

Securely affix the catheter to the patient. Improper securement can result in loss of vascular access.

A firm connection between the catheter hub and administration set is necessary to prevent leaking. The introducer needle is longer than the catheter tube and will enter the vein or artery first. This can cause a blood flashback prior to the entrance of the catheter tube into the vein or artery. Advancing the catheter and needle together slightly will ensure intravascular access for the catheter. When advancing slightly, it is recommended to reduce the angle of approach in order to minimize the chance of puncturing the posterior wall of the vessel.

In the event of an unsuccessful venipuncture, stabilize the catheter hub and actuate the device until it locks in the safety position. The safety position can be found by hearing a slight "click". Remove the catheter and properly dispose of both the spent catheter and locked Safety IV Catheter assembly.

Instruction for use:

Due to the Risk of blood borne pathogen exposure, follow Universal Precautions during placement, use and removal of ClearSafe Comfort® Safety IV Catheter.

Clinicians must be trained in the practice of venipuncture and be aware of inherent dangers.

Keep unshielded needles away from body and fingers at all times.

Asptic technique, proper skin preparation and continued protection of the site areessential. Observe universal precautions on ALL patients.

A. Remove needle cover in a straight outward motion and inspect catheter unit.

B. Grasp device needle hub between thumb and fingers. Verify the release tab and needle bevel are in the up position.

C. Insert needle into skin and vein and observe for blood flashback. The catheter is shorter than the needle, blood flashback may occur before catheter tip is fully in the vein. It may be necessary to advance the catheter and needle slightly to achieve full entry into the vein. To avoid puncturing the posterior vein wall, lower the needle until it is parallel to the skin.

D. Holding the flash chamber stationary, place finger on primary push off tab and thread desired catheter length into vein.

E. Keeping the device stable, retract needle housing until it "clicks" into place. This is the indication that the needle is now locked inside the safety guard.

F. Remove the needle guard by gently twisting and pulling it out of the catheter hub.

G. Connect I.V. Administration line.

H. Immediately discard unit into an approved sharps container.

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